Trilogy Evo, Trilogy Evo O2, Trilogy Evo Universal, & Trilogy EV300

This addendum applies to the Trilogy Evo platform (Trilogy Evo, Trilogy Evo O₂, Trilogy Evo Universal, and Trilogy EV300) and the Clinical and Caregiver manuals. The following sections are supplemental to or replace information within those manuals. The device operator is responsible for reading and understanding this addendum before use.

Contraindications

If the patient has any of the following conditions, consult the patient's health care professional before using noninvasive ventilation:

- An inability to maintain a patent airway or adequately clear secretions
- At risk to aspirate gastric contents
- Acute sinusitis or otitis media
- Epistaxis, causing pulmonary aspiration of blood
- Hypotension

The AVAPS-AE therapy mode is contraindicated for invasive use and patients less than 10 kg.

The AVAPS feature is contraindicated for patients less than 10 kg.

Note: The AVAPS-AE therapy mode and the AVAPS feature are not available for Trilogy Evo Universal devices.

Particulate Filter

The use of the particulate filter is required to protect the device from dirt and dust. Replace at least monthly and between patients.


Note: The filter is printed with the manufacturing date (FAB [MM/YYYY]); the filter should be used within 5 years of that date.

Particulate Filter

Low-Priority System Alarms: Inlet Filter(s) Blocked

Why it occurs	The inlet filter(s) becomes blocked and delivered therapy is reduced.
What to do	<ul style="list-style-type: none"> • Is the air inlet(s) blocked? • Remove, rinse, and reinstall the air-inlet foam filter. Discard and replace the particulate filter. Refer to the instructions in your device manual.
Device performance	The device continues to function.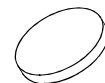

Air-Inlet Foam Filter

Bluetooth Antenna Separation Distance

This product meets RF exposure requirements when it is positioned with a separation distance of at least 20 cm away from the body.

Note: The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Philips Respiration is under license.

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